SINCE 1966
MADE IN ITALY

Aspira

by **FLAEM**®

- I** ISTRUZIONI PER L'USO - Aspiratore professionale medico chirurgico trasportabile
- GB** INSTRUCTIONS FOR USE - Professional medical portable suction machine
- F** INSTRUCTIONS POUR L'UTILISATION - Aspirateur professionnel médico-chirurgical portable
- NL** GEBRUIKSIJNSTRICTIES - Professioneel draagbaar medisch chirurgisch afzuigapparaat
- D** GEBRAUCHSANWEISUNG - Professionelles tragbares Sekretabsauggerät für medizinischen Gebrauch
- E** INSTRUCCIONES PARA EL USO - Aspirador profesional médico quirúrgico portátil
- RUS** ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ - Профессиональный медицинский хирургический портативный аспиратор
- PL** INSTRUKCJA OBSŁUGI - Profesjonalny, chirurgiczny aspirator przenośny
- GR** ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ - Επαγγελματικός μεταφερόμενος αναρροφητήρας για ιατρική και χειρουργική χρήση

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*We are pleased you have purchased our product and we thank you for your trust in us.
It is very important that the health care worker and/or the patient read and understand
the information for use and maintenance.*

INTENDED USE

ASPIRA is an extractor for medical and surgical use. It can be used in health care structures like hospitals, in home care and for pharyngeal aspiration. The device generates suction (aspiration) which allows fluids to be extracted through single use tubing connected to a collection canister which retains the fluids until they can be properly disposed of. Use of the device must be prescribed by a doctor.

Carefully follow the user and maintenance instructions to ensure proper functioning and to extend the life of the device.

ASPIRA (1) is equipped with vacuum flow adjustment knob (4), vacuum gauge (3) and 1000 ml canister (9) with overflow safety device (10) to prevent fluids from entering the suction pump. This device interrupts the vacuum flow via a float-controlled valve found in the cannister lid. It does not require lubrication, is manageable, simple to use, reliable, resistant and quiet. ASPIRA is equipped with the following FLAEM accessories: 1000 ml canister (9) with overflow safety device (10), tubing connection (6/8/14), single use antibacterial/antiviral hydrophobic filter (7), single use aspirator tip (16), single use manual vacuum flow control (15).

Note: Use only original FLAEM accessories

DEVICE NOMENCLATURE

- | | |
|---|-----------------------------------|
| 1) SUCTION MACHINE | 10) SAFETY DEVICE |
| 2) ON/OFF SWITCH | 11) CANISTER LID |
| 3) VACUUM GAUGE | 12) CANISTER "VACUUM" INLET |
| 4) VACUUM FLOW ADJUSTMENT KNOB | 13) CANISTER "PATIENT" INLET |
| 5) AIR INLET | 14) CONNECTING TUBING |
| 6) CONNECTING TUBING | 15) MANUAL VACUUM
FLOW CONTROL |
| 7) (SINGLE USE) HYDROPHOBIC
ANTIBACTERIAL/ANTIVIRAL FILTER (*) | 16) ASPIRATOR TIP |
| 8) CONNECTING TUBING | 17) POWER CABLE |
| 9) COLLECTION CANISTER | |

⚠ IMPORTANT WARNINGS

The manufacturer makes every effort to ensure that every product is of the highest quality and safety; however, as for any electrical device, basic safety regulations must always be followed in order to avoid harming persons and things.

- Before using the product for the first time, and periodically during its lifetime, check the power cable to make sure it is not damaged. In the event it should be damaged, do not plug in the cable and immediately bring the product to an authorised FLAEM service centre or to your trusted dealer.
- In the presence of children and non self-sufficient individuals, the device must be used under the close supervision of an adult who has read this manual.
- Some parts of the device are small enough to be swallowed by children; keep the device out of the reach of children.
- Do not use the provided tubing and cables for any other purpose than those specified, as they can cause risk of strangulation. Be particularly attentive with children and individuals with disabilities because they are often unable to correctly assess risk.
- The suction machine is intended exclusively for the collection of NON-flammable fluids. It is not suitable for use in the presence of a flammable anaesthetic mixture with air, or with oxygen or nitrous oxide.
- Always keep the power cable away from hot surfaces.
- Keep the power cable away from animals (for example, rodents) which could damage the insulation.
- Do not handle the plug with wet hands. Do not use the device in damp conditions (for example when taking a bath or shower). Do not immerse the device in water; in the event of immersion immediately disconnect the plug. Do not remove or touch the immersed device; unplug the power cable first. Immediately bring the device to an authorised FLAEM service centre or to your trusted dealer.
- Use the device only in dust-free conditions, otherwise treatment could be compromised.
- The device casing is not waterproof. Do not wash the device under running water or by immersion and keep it safe from being sprayed by water or other liquids.
- Do not expose the device to particularly extreme temperatures.

- Keep the device away from sources of heat, direct sunlight or excessively warm locations.
- Never obstruct the air inlets positioned on both sides of the device.
- Always operate the device on an unencumbered hard surface.
- Always unplug the power cable immediately after use.
- The device is equipped with an easy to inspect safety fuse in case of malfunction. Before doing so, unplug the power cable. Repairs must be done only by authorised FLAEM personnel. Unauthorised repairs void the warranty and may be hazardous for the user.

ATTENTION: Do not modify this device without authorisation from the manufacturer.

- The Manufacturer, the Vendor and the Importer shall be held responsible for safety, reliability and performance only if: a) the device is used in compliance with the instructions for use b) the wiring where the device is being used is in compliance with safety regulations and current laws.
- The Manufacturer must be contacted to communicate unexpected problems/events related to functioning.
- Ensure that the connection tubing and the canister lid have been carefully sealed in order to avoid loss of suction.
- Do not upset the canister while the device is working as the liquid may be aspirated into the device, damaging the pump. Should this occur, immediately switch off the suction machine, then empty and clean the canister (send it to an authorised FLAEM service centre).
- When the safety device (10) intervenes, suction is halted. Switch off the device, empty the canister (9) and carry out the cleaning procedure.
- We recommend personal use of accessories, collection canisters and connection tubing to prevent the risk of contagious infection.
- The aspirator tip and the manual vacuum flow control are single use sterilised products and must be replaced after each use.
- Check the expiration date on the original aspirator tip packaging and make sure the sterile packaging is intact. FLAEM NUOVA declines any responsibility for harm to the patient correlated to the deterioration of the aforementioned sterile packaging due to third party handling during the original packaging of the entire device.
- (*)The hydrophobic antibacterial/antiviral filter is **single use**. The filter used by the same patient must be replaced every 24 hours. ATTENTION: the hydrophobic antibacterial/antiviral filter must be replaced for each new patient.

INSTRUCTIONS FOR USE


Before each use, the accessories must be carefully inspected to ensure the absence of dust, incrustation, clots or liquid substances both inside the connecting tubing and in the canister and its respective lid. Furthermore, they must be cleaned following the instructions rigorously as stated in the "CLEANING, SANITISATION, DISINFECTION, STERILISATION" paragraph. We recommend personal use of the accessories, the collection canisters and the connecting tubing to prevent risk of contagious infection.

Instructions for suction on the patient:

1. Connect the accessories, referring to the "CONNECTION DIAGRAM" on the cover.

ATTENTION: in addition to being antibacterial/antiviral, the hydrophobic filter also acts as secondary line of safety and stops any fluids that the primary safety device in the canister lid accidentally does not stop.

Follow the directions below for proper assembly: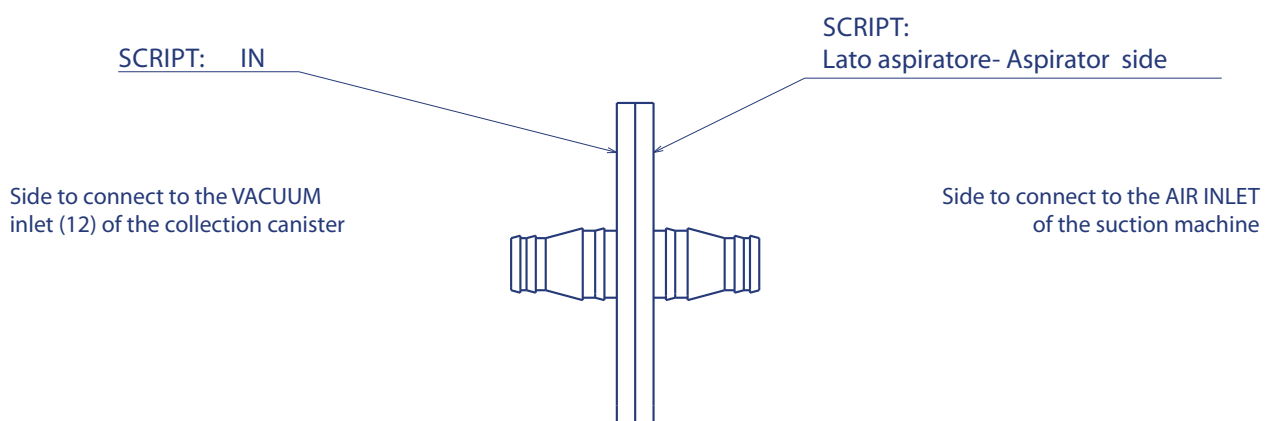

2. Plug in the power cable (17) to a mains network socket which corresponds to the electrical data of the device. The position of the socket must be such that the device can be easily unplugged from the mains network.
3. Switch on the device using the on/off switch (2), keeping in mind that the device is intended to be used on a horizontal surface in order for the safety device (10) to function correctly and prevent liquids from entering the suction pump. The device must be kept away from walls to allow for adequate engine cooling.
4. For more comfortable aspiration, set the desired vacuum value (bar) using the vacuum adjustment knob(4). Vacuum value is increased by turning the knob clockwise and decreased by turning the knob counter-clockwise; these values can be read on the "vacuum gauge" (3). To accelerate aspiration and simplify cleaning, we recommend putting approximately 400 ml of water in the canister (16).
5. Place your finger on the manual vacuum flow control (15) and, activating it in pulses, begin suction on the patient using the aspirator tip.
6. Once the process is complete, switch off the device.
7. Remove the plug, then empty and clean the canister and connecting tubing.

CLEANING, SANITISATION, DISINFECTION, STERILISATION



Switch off the device before any cleaning procedure and unplug the power cable from the socket.

DEVICE CLEANING


Use only a damp cloth with antibacterial soap (non-abrasive and with no solvents of any sort).

COLLECTION CANISTER AND CONNECTING TUBING


- Detach the aspirator tip (16), the manual vacuum flow control (15) and the tubing (14) from the canister (9). Disconnect the tubing (6/8) from both the canister and the filter (7). Remove the canister from its holder, keeping it upright, and empty it (in the WC at home, in the biological waste container in the hospital) and clean it, disassembling it as shown below: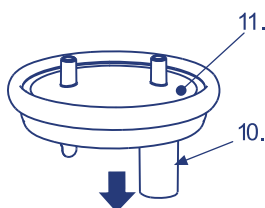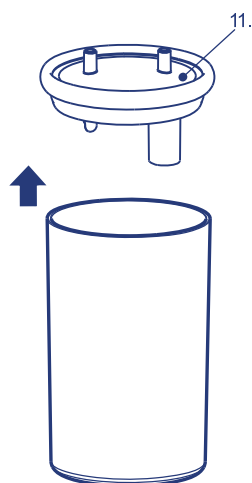

1. Remove the safety device (.10) from the canister lid (.11)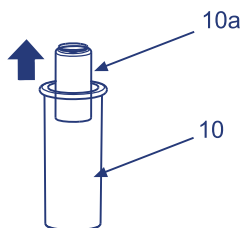

2. Remove the float (.10a) from the safety device (.10)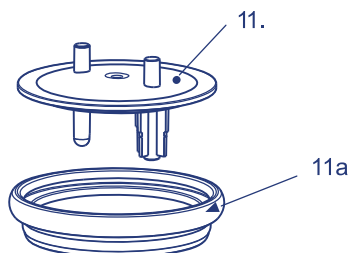

3. Remove the silicone gasket (11a) from the canister lid (.11)

SANITISATION

Before and after each use, sanitise the collection canister and the connecting tubing choosing one of the methods described below.

(method A): rinse each part (6,8,9,10,10a,11,11a,14) with hot potable water (approximately 40°C) with gentle dish washing soap (non-abrasive) or in the dishwasher set to a hot cycle

(method B): immerse each part (6,8,9,10,10a,11,11a,14) in a solution of 60% water and 40% white vinegar. Lastly, rinse abundantly with hot potable water (approximately 40°C).

(method C): boil each of the parts (6,8,9,10,10a,11,11a,14) in water for 20 minutes; it is preferable to use de-mineralised or distilled water to prevent calcium deposits.

After having sanitised the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, a hair dryer).

DISINFECTION

The accessories which can be disinfected are (6,8,9,10,10a,11,11a,14)

The disinfection procedure described in this paragraph is to be followed before using the accessories. It is effective on the parts that undergo this treatment only if each step is followed and only if the parts to be treated have previously been sanitised.

The disinfectant must be an electrolytic chloroxidizer (active principle: sodium hypochlorite) specific for disinfecting, which is available in any pharmacy.

Implementation:

- Fill a container big enough to hold all of the parts to disinfect with a solution of potable water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
- Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant associated with the concentration chosen for the solution.
- Remove the disinfected parts and rinse abundantly with warm potable water.
- After having disinfected the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, a hair dryer).
- Dispose of the solution following the instructions provided by the disinfectant manufacturer.

STERILISATION

The accessories that can be sterilised are (6,8,9,10,10a,11,11a,14)

The sterilisation procedure described in this paragraph is effective on the parts that undergo this treatment only if every step is followed and only if the parts to be treated have previously been sanitised. The procedure is validated in its conformity to ISO 17665-1.

Device: Fractionated vacuum overpressure steam steriliser in accordance with EN 13060.

Implementation: Wrap every single part to be treated with a sterile barrier system or packaging in accordance with Norm EN 11607. Place the wrapped parts into the steam steriliser, ensuring that the canister (9) is in an up-right position.

Following the device instruction manual, start the sterilisation cycle and select a temperature of:

- 134°C and a duration of 10 minutes for a maximum of 30 times for the SILICONE TUBING (6,8,14) AND the GASKET (11a)
- 121°C and a duration of 15 minutes for a maximum of 50 times for the POLYCARBONATE CANISTER (9).

Storage: Store the sterilised parts as per the instructions for use of either the sterile barrier system or packaging. After sanitising, disinfecting or sterilising, reassemble the canister and the connecting tubing following the directions provided in the "CONNECTION DIAGRAM".

- The aspirator tip and the manual vacuum flow control are single use sterilised products and must be replaced after each use.
- ATTENTION : the antibacterial/antiviral filter (7) is hydrophobic and **single use**, it must be replaced for every new patient . Should the suction machine be used only by one patient, the filter must be replaced every 24 hours.

TROUBLE-SHOOTING

Switch off the device before any procedure and unplug the power cable from the socket.

PROBLEM	CAUSE	SOLUTION
The device does not work	The power cable has not been correctly inserted into the socket of the device or the power socket	- Correctly insert the power cable into the sockets
Lack of suction	The collection cannister lid has not been correctly positioned onto the cannister	- Correctly position the collection cannister lid
	Lid gasket not in place	- Correctly position the gasket on the lid
Lack of suction caused by fluid leakage	Blocked filter	- Replace the filter
Blocked float	Incrustation on the float	- Remove the cannister lid and safety device and extract the float. Continue by carrying out the cleaning procedures as described in the paragraph, "CLEANING, SANITISATION, DISINFECTION, STERILISATION"
Vacuum power poor and/or non-existent	Vacuum adjustment knob completely open	- Fully close the adjustment knob and check the vacuum power
	Blocked protection filter	- Replace the filter
	Connecting tubing to the filter and device clogged, bent or disconnected	- Check the condition of the tubing, replace it if blocked and correctly connect it as per the "ASSEMBLY DIAGRAM" of this manual
	Cannister lid overflow valve closed or blocked	- Unblock the overflow valve, keeping the device upright
	Dirty, blocked or damaged pump	- Take the device to your trusted dealer or an authorised FLAEM service centre

If, after checking the above-described conditions the device is still not working properly, we recommend taking it to your nearest trusted dealer or authorised FLAEM service centre.

ACCESSORY SPECIFICATIONS

- single use hydrophobic antibacterial/antiviral filter
- PC collection canister w/ compl. lid
- Ø 13 x 7.5 mm L 370 mm silicone tubing
- Ø 13 x 7.5 mm L 1300 mm silicone tubing
- Ø 13 x 7.5 mm L 80 mm silicone tubing
- CH18 single use sterile aspirator tip
- manual vacuum flow control (single use)

MICROBIAL CONTAMINATION

In the presence of pathologies with the risk of infection and microbial contamination, we recommend personal use of the accessories, collection canister and connecting tubing (consult your doctor).


Interactions:

The materials used for contact with the secretions are highly stable and chemical resistant thermoplastic polymers (PP, PC, SI). We cannot, however, exclude interactions. Therefore, it is suggested: a) to always avoid prolonged contact of liquid with the canister or tubing and sanitise immediately after use. b) Should anomalous situations occur, i.e. softening or cracking of the accessories, quickly terminate the procedure and substitute the used materials. Contact the authorised service centre and specify how the product was used.

Note: Use only with original FLAEM accessories.

CANISTER HOLDER

For convenience, the canister holder can be detached and reattached on the most suitable side (left or right) as shown in the figure:

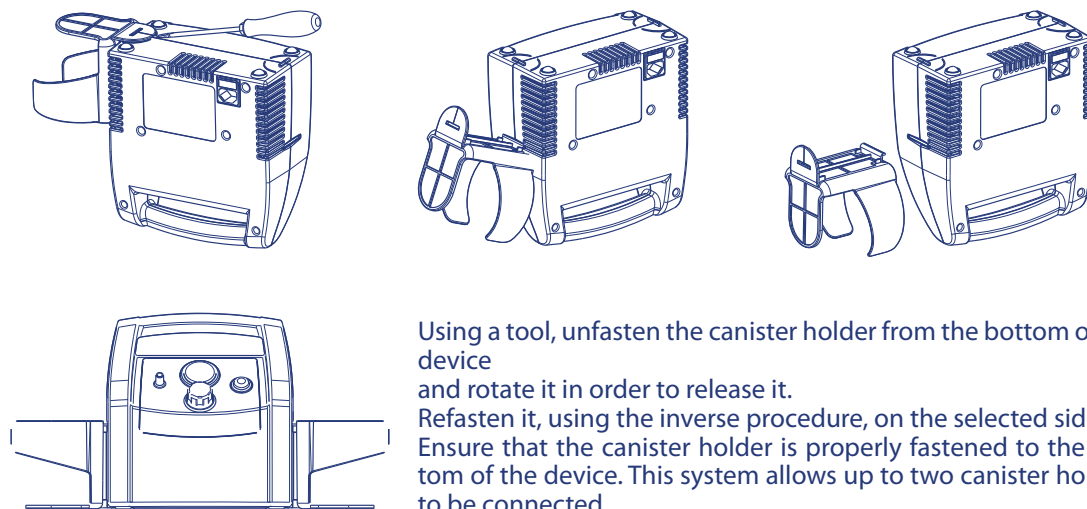Using a tool, unfasten the canister holder from the bottom of the device and rotate it in order to release it. Refasten it, using the inverse procedure, on the selected side. Ensure that the canister holder is properly fastened to the bottom of the device. This system allows up to two canister holders to be connected.

TECHNICAL SPECIFICATIONS

Mod. P1211EM - Mod. P1211EM Plus

Non-lubricated rotary piston suction pump with thermal protection.

Power source/power	230 V ~ 50 Hz / 210 VA
Fuse	1 x T2A 250V
Device dimensions	32.5 (L) x 12 (W) x 23.5 (H) cm
Weight	2.4 Kg
Noise (at 1 m)	56 dB (A) (approx.)
Use	Continuous

In accordance with EEC Directive 93/42  0051

APPLIED PARTS

Type BF applied parts are:

Operating conditions:

patient accessories (16)
Temperature min. 5°C; max. 35°C
Relative humidity min. 10%; max. 95%

Storage conditions:

Temperature min. -25°C; max 70°C
Relative humidity min. 10%; max. 95%

Operating/storage
atmospheric pressure:

min. 690 hPa; max. 1060 hPa

	Mod. P1211EM	Mod. P1211EM Plus
Suction:	high vacuum / low flow	high vacuum / high flow
Adjustable vacuum level:	from -0,05 a -0,86 bar approx (accuracy class 2.5)	from -0,05 a -0,82 bar approx (accuracy class 2.5)
Max air flow:	approx 13.5 l/min	approx 20 l/min

SYMBOLS

	Class II device		Risk: electrocution. Consequence: Death. Do not use the device while taking a bath or a shower
	Type BF applied part		"ON"
	Attention: check the instructions for use		"OFF"
	Single use	When the device is switched off, the on/off switch stops compressor function in only one phase of the two-phase power supply.	
	Alternating current		EEC ref. Directive 93/42 CE medical mark and subsequent revisions
	Ethylene oxide sterilisation		Without latex
SN	Device serial number		Production year
	TÜV Certification ref. ISO 10079-1		Manufacturer

DEVICE DISPOSAL

In compliance with the Directive 2002/96/CE, the symbol printed on the device shows that the device to be disposed of is considered waste and must therefore be an item of "differentiated collection". Consequently, the user must take it (or have it taken) to the differentiated collection sites provided by the local authorities, or turn it in to the dealer when purchasing an equivalent new device. Differentiated waste collection and the subsequent treatment, recycling and disposal procedures promote the production of devices made with recycled materials and limit the negative effects on the environment and on health caused by potential improper waste management. Unauthorised disposal of the product by the user shall result in the application of administrative penalties, of which article 50 and the subsequent revisions of Italian Legislative Decree no. 22/1997.

ELECTROMAGNETIC COMPATIBILITY

This device was designed to satisfy the currently required requisites for electromagnetic compatibility (EN 60 601-1-2:2007). Electro-medical devices require particular care during installation and use relative to EMC requirements. Users are therefore requested to install and/or use these devices following the manufacturer's specifications. There is a risk of potential electromagnetic interference with other devices, in particular with other analysis and treatment devices. RF mobile or portable radio and telecommunications devices (mobile telephones or wireless connections) can interfere with the functioning of electro-medical devices. For further information visit our website www.flaemnuova.it.

Flaem reserves the right to make technical and functional modifications to the product with no prior warning.

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© Copyright by FLAEM NUOVA S.p.A., 2011