# RESMED

# **Swift<sup>™</sup> FX Bella**


+ Swift™ FX for Her HEADGEAR


NASAL PILLOWS SYSTEM

# **User Guide**



# RESMED

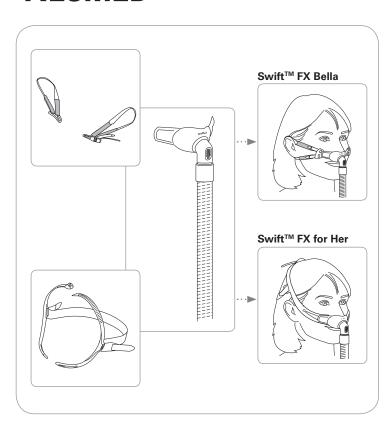

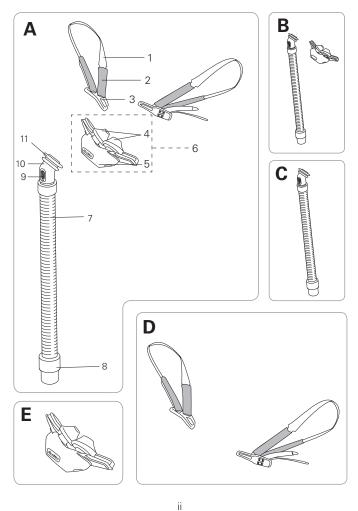

# Mask components / Maskenkomponenten / Composants du masque / Componenti della maschera / Maskeronderdelen

Item / Artikel / Désignation / Parte / Onderdeel	Description / Beschreibung / Description / Descrizione / Beschrijving	Part number / Teilenummer / Code produit / Numero parte / Onderdeelnummer
1	Strap / Band / Sangle / Cinghia / Band	_
2	Soft sleeve / Komforthüllen / Gaine souple / Guaina morbida / Zachte hoes	_
3	Buckle / Clip / Boucle / Fibbia / Gesp	_
4	Nasal pillows / Nasenpolster / Coussins narinaires / Cuscinetti nasali / Neuskussentjes	_
5	Cushion / Maskenkissen / Bulle / Cuscinetto / Kussentje	_
6	Pillows / Polster / Coussins / Cuscinetti / Kussentjes	_
7	Short tube / Verbindungsschlauch / Tuyau court / Tubo corto / Korte slang	_
8	Swivel / Drehadapter / Pièce pivotante / Giunto rotante / Draaibare kop	_
9	Vent / Luftauslassöffnung / Orifice de ventilation / Foro per l'esalazione / Ventilatieopening	_
10	Elbow / Kniestück / Coude / Gomito / Bocht	_
11	Swivel ring / Drehadapterring / Anneau pivotant / Anello rotante / Ring draaibare kop	_

Item / Artikel / Désignation / Parte / Onderdeel	Description / Beschreibung / Description / Descrizione / Beschrijving	Part number / Teilenummer / Code produit / Numero parte / Onderdeelnummer
A	Swift FX Bella complete system / Swift FX Bella Komplettsystem / Masque Swift FX Bella complet / Cuscinetti nasali Swift FX Bella con tutti i componenti / Volledig Swift FX Bella-systeem	61561 EUR 1* 61564 ANZ**
В	Pillows system (Frame system) / Nasenpolstermaske (Maskenhaltersystem) / Masque narinaire (support complet) / Cuscinetti nasali (Telaio) / Kussentjessysteem (framesysteem)	61510 (XS) 61511 (S) 61512 (M) 61513 (L)
С	Short tube assembly (Frame assembly) / Verbindungsschlauch (Maskenhalter) / Ensemble tuyau court (ensemble support) / Gruppo circuito corto (Complesso del telaio) / Korte slang-assemblage (frame-assemblage)	61528 (1) 61527 (10)
D	Swift FX Bella headgear assembly / Swift FX Bella Kopfband / Ensemble harnais Swift FX Bella / Gruppo del copricapo Swift FX Bella / Assemblage Swift FX Bella- hoofdbandenstel	61581 (1 x P) / 61581 (1 x P) / 61581 (1 x R) / 61581 (1 x R) / 61581 (1 x R)
Е	Pillows / Polster / Coussins / Cuscinetti / Kussentjes	61520 (XS) 61521 (S) 61522 (M)

Item / Artikel / Désignation / Parte / Onderdeel	Description / Beschreibung / Description / Descrizione / Beschrijving	Part number / Teilenummer / Code produit / Numero parte / Onderdeelnummer
	/ Außerdem erhältlich: / Égalemen Tevens verkrijgbaar	t disponible /
	Pillows / Polster / Coussins / Cuscinetti / Kussentjes	61523 (L)
	Swift FX Bella + Swift FX for Her headgear (Combination pack) / Swift FX Bella + Swift FX for Her Kopfband (Kombipackung) / Harnais Swift FX Bella + Swift FX for Her (kit combiné) / Copricapi Swift FX Bella + Swift FX for Her (Combinazione) / Swift FX Bella- + Swift FX for Her-hoofdbandenstel (combinatieverpakking)	61580


XS Extra Small, S Small, M Medium, L Large / XS Extra Small, S Small, M Medium, L Large / XS: très petit, S: petit, M: médium, L: grand / XS Extra Small, S Small, M Medium, L Large / XS Extra Small, S Small, M Medium, L Large P Pink / P Pink / P: rose / R Rosa / R Roze

<sup>\*</sup>EUR 1: France, Germany, UK only / Nur Frankreich, Deutschland, Großbritannien / Allemagne, France et Royaume-Uni uniquement / disponibile solo in Francia, Germania e Regno Unito / Alleen Frankrijk, Duitsland en Verenigd Koninkrijk

<sup>\*\*</sup>ANZ: Australia, New Zealand only / Nur Australien, Neuseeland / Australie et Nouvelle-Zélande uniquement / disponibile solo in Australia e Nuova Zelanda / Alleen Australië en Nieuw-Zeeland


# Fitting / Anlegen / Mise en place / Applicazione / Opzetten



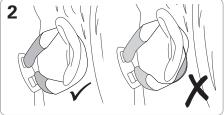





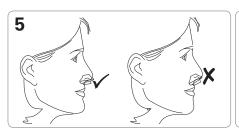



# Disassembly / Auseinanderbau / Démontage / Smontaggio / Demontage













# Reassembly / Zusammenbau / Remontage / Riassemblaggio / Opnieuw monteren

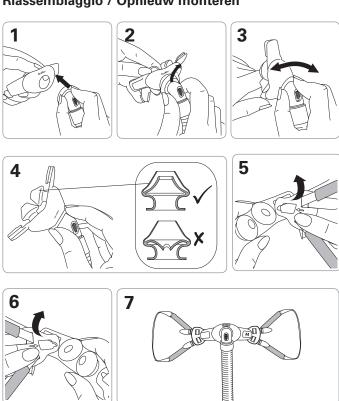


Reassembly of straps (if required) / Wiederanbringen der Bänder (falls erforderlich) / Remontage des sangles (le cas échéant) / Riassemblaggio delle cinghie (se richiesto) / Opnieuw monteren van banden (indien nodig)

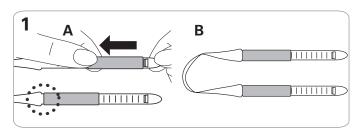

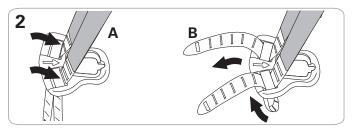

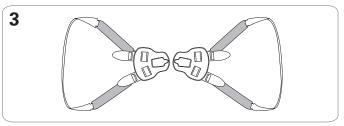

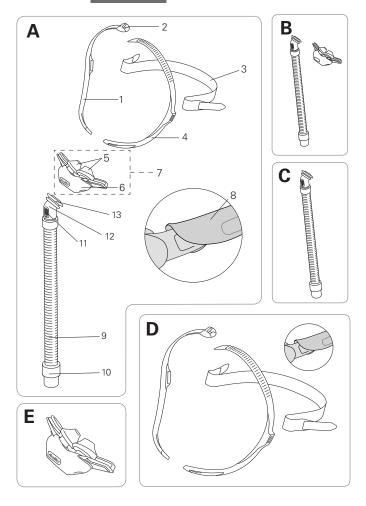



# Mask components / Maskenkomponenten / Composants du masque / Componenti della maschera / Maskeronderdelen

Item / Artikel / Désignation / Parte / Onderdeel	Description / Beschreibung / Description / Descrizione / Beschrijving	Part number / Teilenummer / Code produit / Numero parte / Onderdeelnummer
1	Headgear right / Kopfband rechts / Harnais droit / Parte destra copricapo / Rechterkant hoofdband	_
2	Top buckle / Oberer Clip / Boucle supérieure / Fibbia superiore / Bovenste hoofdbandgesp	_
3	Back strap (1 x P) / Hinteres Band (1 x P) / Sangle arrière (1 x P) / Cinghia posteriore (1 x R) / Achterste band (1 x R)	_
4	Headgear left / Kopfband links / Harnais gauche / Parte sinistra copricapo / Linkerkant hoofdband	_
5	Nasal pillows / Nasenpolster / Coussins narinaires / Cuscinetti nasali / Neuskussentjes	_
6	Cushion / Maskenkissen / Bulle / Cuscinetto / Kussentje	_
7	Pillows / Polster / Coussins / Cuscinetti / Kussentjes	_
8	Soft wraps / Weiche Hüllen / Housses souples / Involucri imbottiti / Zachte beschermers	61544 (2 x P) / 61544 (2 x P) / 61544 (2 x P) / 61544 (2 x R) / 61544 (2 x R)
9	Short tube / Verbindungsschlauch / Tuyau court / Tubo corto / Korte slang	_
10	Swivel / Drehadapter / Pièce pivotante / Giunto rotante / Draaibare kop	_

# Swift FX for Her

Item / Artikel / Désignation / Parte / Onderdeel	Description / Beschreibung / Description / Descrizione / Beschrijving	Part number / Teilenummer / Code produit / Numero parte / Onderdeelnummer
11	Vent / Luftauslassöffnung / Orifice de ventilation / Foro per l'esalazione / Ventilatieopening	_
12	Elbow / Kniestück / Coude / Gomito / Bocht	_
13	Swivel ring / Drehadapterring / Anneau pivotant / Anello rotante / Ring draaibare kop	_
A	Swift FX for Her complete system / Swift FX for Her Komplettsystem / Masque Swift FX for Her complet / Cuscinetti nasali Swift FX for Her con tutti i componenti / Volledig Swift FX for Her-systeem	61545 EUR 1 61546 EUR 2 61547 EUR 3
В	Pillows system (Frame system) / Nasenpolstermaske (Maskenhaltersystem) / Masque narinaire (support complet) / Cuscinetti nasali (Telaio) / Kussentjessysteem (framesysteem)	61510 (XS) 61511 (S) 61512 (M) 61513 (L)
С	Short tube assembly (Frame assembly) / Verbindungsschlauch (Maskenhalter) / Ensemble tuyau court (ensemble support) / Gruppo circuito corto (Complesso del telaio) / Korte slang-assemblage (frame-assemblage)	61528 (1) 61527 (10)
D	Swift FX for Her headgear assembly / Swift FX for Her Kopfband / Ensemble harnais Swift FX for Her / Gruppo del copricapo Swift FX for Her / Assemblage Swift FX for Her-hoofdbandenstel	61543

#### Swift FX for Her

Item / Artikel / Désignation / Parte / Onderdeel	Description / Beschreibung / Description / Descrizione / Beschrijving	Part number / Teilenummer / Code produit / Numero parte / Onderdeelnummer
Е	Pillows / Polster / Coussins / Cuscinetti / Kussentjes	61520 (XS) 61521 (S) 61522 (M)
Also available / Außerdem erhältlich: / Également disponible / Altri prodotti /Tevens verkrijgbaar		
	Pillows / Polster / Coussins / Cuscinetti / Kussentjes	61523 (L)

XS Extra Small, S Small, M Medium, L Large / XS Extra Small, S Small, M Medium, L Large / XS: très petit, S: petit, M: médium, L: grand / XS Extra Small, S Small, M Medium, L Large /


P Pink / P Pink / P : rose / R Rosa / R Roze

XS Extra Small, S Small, M Medium, L Large


# Fitting / Anlegen / Mise en place / Applicazione / Opzetten





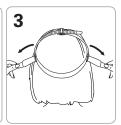




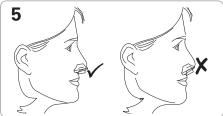



# Disassembly / Auseinanderbau / Démontage / Smontaggio / Demontage




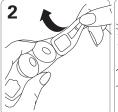








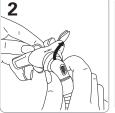



# Reassembly / Zusammenbau / Remontage / Riassemblaggio / Opnieuw monteren

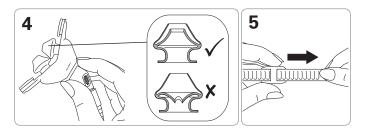









#### **English**

# Swift<sup>™</sup> FX Bella + Swift FX for Her HEADGEAR

#### NASAL PILLOWS SYSTEM

Thank you for choosing the Swift FX Bella + Swift FX for Her Headgear. The Swift FX Bella + Swift FX for Her Headgear is the first of ResMed's Nasal Pillows Systems that offers a choice of two different types of headgear. This user guide provides you with instructions for setting up and using both headgear with your nasal pillows system.

#### Intended use

The Swift FX Bella and Swift FX for Her channels airflow noninvasively to a patient from a positive airway pressure (PAP) device such as a continuous positive airway pressure (CPAP) or bilevel system.

The Swift FX Bella and Swift FX for Her are:

- to be used by adult patients (30 kg) for whom positive airway pressure has been prescribed
- intended for single-patient re-use in the home environment and multipatient re-use in the hospital/institutional environment.

#### Using your mask

When using your mask with ResMed CPAP or bilevel devices that have mask setting options, refer to the Technical specifications section in this user guide for mask selection options.

For a full list of compatible devices for these masks, see the Mask/ Device Compatibility List on www.resmed.com on the **Products** page under **Service & Support**. If you do not have internet access, please contact your ResMed representative.

#### Notes:

- These masks are not compatible for use with ResMed AutoSet CS™ 2 and VPAP™ Adapt SV devices.
- SmartStop may not operate effectively when using these masks with some CPAP or bilevel devices that have this feature.
- If you experience nasal dryness or irritation, use of a humidifier is recommended



#### **GENERAL WARNINGS**

- The vent holes must be kept clear.
- The mask should only be used with CPAP or bilevel devices recommended by a physician or respiratory therapist.
- The mask should not be used unless the device is turned on. Once the mask is fitted, ensure the device is blowing air. Explanation: CPAP and bilevel devices are intended to be used with special masks (or connectors) which have vent holes to allow continuous flow of air out of the mask. When the device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask vent holes. However, when the device is not operating, insufficient fresh air will be provided through the mask, and the exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This applies to most models of CPAP or bilevel devices.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the CPAP or bilevel device is not operating, so that unused oxygen does not accumulate within the CPAP or bilevel device enclosure and create a risk of fire.
- Oxygen supports combustion. Oxygen must not be used while smoking or in the presence of an open flame. Only use oxygen in well ventilated rooms.

- At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration varies, depending on the pressure settings, patient breathing pattern, mask, point of application and leak rate. This warning applies to most types of CPAP devices.
- The technical specifications of the mask are provided for your clinician to check that they are compatible with the CPAP or bilevel device. If used outside specification or if used with incompatible devices, the seal and comfort of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect the CPAP or bilevel device function.
- Discontinue using the mask if you have ANY adverse reaction to the use of the mask, and consult your physician or sleep therapist.
- Using a mask may cause tooth, gum or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.
- As with all masks, some rebreathing may occur at low CPAP pressures.
- Refer to your CPAP or bilevel device manual for details on settings and operational information.
- Remove all packaging before using the mask.



#### **Fitting**

The Swift FX Bella fitting illustrations in this user guide show the sequence for obtaining the best fit and comfort when using your mask.

#### Fitting steps

- Hold the pillows at your nose ensuring that the ResMed logo is on the top of the pillows facing up. Pull one strap around and behind your ear.
- 2. Ensure the strap lies flat and is not twisted.
- Repeat steps 1 and 2 with the other strap.
- Adjust the mask fit by pulling the upper straps along the notches until a comfortable fit is achieved. Repeat with the lower straps. Do not overtighten.
- Check that the nasal pillows are positioned upright in the entrance of your nostrils.
- Finally, connect the free end of the short tube assembly to the device air tubing. The mask is now fitted and ready for use



#### **WARNING**

Using Swift FX Bella may cause ear soreness. If symptoms occur, consult your physician.

#### Removing your mask

To remove your mask, place your fingers underneath the straps and lift them away from your ears.

#### Disassembly

Refer to the Swift FX Bella disassembly illustrations in this user guide.

#### Disassembly steps

- 1–2. Remove each buckle by slightly twisting the buckle and pulling away from the node on the pillows.
- 3–4. Gently push the swivel ring up and pull out of the cushion.

**Note:** The strap and soft sleeve are not intended to be dissassembled and can be washed together, while attached to the buckle.

### Reassembly

Refer to the Swift FX Bella reassembly illustrations in this user guide.

#### System reassembly steps

- 1–2. Insert the swivel ring into the opening of the cushion. Ensure that the cushion sits between the ridges of the swivel ring.
- Rotate the elbow to ensure it moves freely.
- Check that the inner walls of the nasal pillows are not compressed or collapsed. If so, squeeze each nasal pillow to release the inner wall.
- Hold the buckle with the arrow facing you. Latch the buckle onto the node on the pillows.
- 6. Repeat step 5 with the other buckle.

#### Headgear reassembly steps (if required)

If you have disassembled your headgear it needs to be reassembled before attaching to the pillows.

- Thread each strap through the soft sleeve. Pull the soft sleeve up until it sits flush against the widest part of the strap.
   Ensure the soft sleeve lies flat on the strap.
- 2. Hold the buckle with the arrow facing up.
  - A. Ensuring the notches of the strap are facing the arrow head, thread the end of the strap down through the opening closest to the arrow head on the buckle.
  - B. Bring the end of the strap up through the other opening, making sure that the notches on the strap sit against each other.
  - Repeat these steps with each strap.
- Ensure that the straps are not twisted.

### **Fitting**

The Swift FX for Her fitting illustrations in this user guide show the sequence for obtaining the best fit and comfort when using your mask.

#### Fitting steps

- Hold the pillows at your nose ensuring that the ResMed logo is on the top of the pillows facing up. Pull the headgear over your head. The headgear should sit midway between the ears and eyes.
- Adjust the top straps by pulling the strap through the buckle, one notch at a time. Do not overtighten.
- Adjust the side straps until a comfortable fit is achieved. The back strap can be positioned over or under the hair.
- Adjust the headgear until the nasal pillows sits comfortably in your nose.
- Check that the nasal pillows are positioned upright in the entrance of your nostrils.
- Finally, connect the free end of the short tube assembly to the device air tubing.

The mask is now fitted and ready for use.

### Removing your mask

To remove your mask loosen the upper strap, then pull the mask and headgear up over your head.

### Disassembly

Refer to the Swift FX for Her disassembly illustrations in this user guide.

#### Disassembly steps

- 1-2. Remove the headgear by slightly twisting each headgear arm and pulling away from the node on the pillows.
- 3-4. Gently push the swivel ring up and pull out of the cushion.

### Reassembling

Refer to the Swift FX for Her reassembly illustrations in this user guide.

#### Reassembly steps

- 1–2. Insert the swivel ring into the opening of the cushion. Ensure that the cushion sits between the ridges of the swivel ring.
- 3. Rotate the elbow to ensure it moves freely.
- Check that the inner walls of the nasal pillows are not compressed or collapsed. If so, squeeze each nasal pillow to release the inner wall.
- 5. Thread the headgear strap through the top buckle.
- Hold the headgear arm with the recess facing you. Latch the headgear arm onto the node on the pillows.
- 7. Repeat step 6 with the other headgear arm.
- 8–9. Thread the back strap through each side of the headgear, then fold over to secure. Ensure that the hook tab is on the outside.

#### Cleaning your mask in your home

Your mask and headgear should only be handwashed by gently rubbing in warm (approximately 30°C) water using mild soap. All components should be rinsed well with drinking quality water and allowed to air dry out of direct sunlight.



## WARNING

Do not use aromatic-based solutions or scented oils (eg, eucalyptus or essential oils), bleach, alcohol or products that smell strongly (eg, citrus) to clean any of the mask components. Residual vapours from these solutions can be inhaled if not rinsed thoroughly. They may also damage the mask, causing cracks.



# CAUTION

- If any visible deterioration of a system component is apparent (cracking, crazing, tears or cushion damage), the component should be discarded and replaced.
- Avoid connecting flexible PVC products (eg, PVC tubing) directly to any part of the mask. Flexible PVC contains elements that can be detrimental to the materials of the mask, and may cause the components to crack or break.

#### Daily/After each use:

- To optimise the mask seal, facial oils should be removed from the cushion after use.
- Handwash the separated mask components (excluding headgear).
- If the vent requires cleaning use a soft bristle brush.

#### Weekly:

Handwash the headgear. It may be washed without being disassembled.

### Reprocessing the mask between patients

This mask should be reprocessed when used between patients. Cleaning, disinfection and sterilisation instructions are available from the ResMed website, www.resmed.com/masks/sterilization.If you do not have internet access, please contact your ResMed representative.

# **Troubleshooting**

iroubleshooting			
Problem / possible cause	Solution		
Pillows won't seal properly, are uncomfortable or cause red marks			
Pillows may have been fitted incorrectly, adjusted incorrectly or the headgear is too tight.	Check that the ResMed logo on top of the pillows is facing outwards. Carefully follow the fitting instructions. Readjust the headgear to ensure it is not over tightened.		
Inner wall position of one or both pillows is incorrect.	Squeeze pillow sides to release inner wall. The correct position is shown in the reassembly instructions.		
Pillows are the wrong size.	Talk to your clinician.		
Pillows may be dirty.	Clean pillows according to the instructions.		
Mask is too noisy			
System is assembled incorrectly.	Disassemble the mask, then reassemble according to the instructions.		
System is leaking air.	Refit or reposition you mask. Adjust your headgear. Check your mask is assembled correctly.		
Vent is blocked or partially blocked.	If the vent requires cleaning, use a soft bristle brush.		

Swift FX Bella Sv	vift FX for Her			
Problem / possible	e cause	Solution		
Ears are sore				
Swift FX Bella head fitted correctly.	lgear is not	Readjust the strap are not overtighter If problem persists clinician.	ned or twi	sted.
Swift FX Bella headgear is not suited for your ear type.  Replace the Swift FX Bella headgear with the supplied Swift FX for Her headgear.		Swift		
Technical spe	cifications			
Pressure–flow curve		ins passive venting to a result of manufactu av vary.		
© 00			Pressure (cm H <sub>2</sub> O)	Flow (L/min)
vent flow rate (L/min)			4	20
g 30			8	29
No 20			12	37
Vent	0 2		16	43
4 6 8	Mask pressure (cm H <sub>2</sub> )	6 8 20 O)	20	49
Dead space information		pace is the empty volue e swivel. Using the la		
Therapy pressure	4 to 20 cm H <sub>2</sub> O			
Resistance	Drop in pressure at 50 L/min: 0.4 at 100 L/min: 1.4			

1. 00 110	
Therapy pressure 4 to 20 cm H <sub>2</sub> O	
Resistance Drop in pressure measured (nominal) at 50 L/min: 0.4 cm H <sub>2</sub> O at 100 L/min: 1.4 cm H <sub>2</sub> O	
Environmental Operating temperature: +5°C to +40°C  Operating humidity: 15% to 95% relative humidit  non-condensing Storage and transport temperature: -20°C to +60° Storage and transport humidity: up to 95% relative humidity non-condensing	C

lechnical specifications		
Sound	DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is 25 dBA, with uncertainty 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1m is 17 dBA, with uncertainty 3 dBA.	
Gross dimensions	Large: 415 mm (H) x 94 mm (W) x 45 mm (D) Mask fully assembled with short tube assembly (no headgear)	
Mask setting options	For S9: Select 'Pillows' For other devices: Select 'SWIFT' (if available),	

#### Notes:

- The mask system does not contain PVC, DEHP or phthalates.
- This product does not contain natural rubber latex.
- The manufacturer reserves the right to change these specifications without notice

otherwise select 'MIRAGE' as the mask option.

### **Storage**

Ensure that the mask is thoroughly clean and dry before storing it for any length of time. Store the mask in a dry place out of direct sunlight.

### **Disposal**

The mask does not contain any hazardous substances and may be disposed of with your normal household refuse.

# **Symbols**

Caution, consult accompanying documents; Cot Batch code;

REF Catalogue number; Temperature limitation; Manufacturer; Manufactur

#### **Consumer Warranty**

ResMed acknowledges all consumer rights granted under the EU Directive 1999/44/EG and the respective national laws within the EU for products sold within the European Union.



#### Manufacturer:

ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. **Distributed by:** 

ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA.

[EC | REP] ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire 0X14 4RY UK.

See www.resmed.com for other ResMed locations worldwide.

For patent information, see www.resmed.com/ip Swift is a trademark of ResMed Ltd and is registered in U.S. Patent and Trademark Office.

© 2012 ResMed Ltd.























618252/1 2012-06 SWIFT FX Bella USER

