

PARI *chamber*

ANTISTATIC

Grows with you

Regardless of age, if at home or on the go, PARIchamber is the VHC for all your patients

Key features

- Antistatic valved holding chamber (VHC) for reliable metered dosing
- Clear mouthpiece for a visual check with a low resistant expiratory valve
- Universal MDI adapter
- Can be disinfected in boiling water or in a steam disinfectant
- *SmartTouch*® masks adaptable to any application



www.pari.de



0–2 years with baby mask Ladybug Item no. 035G2020	Above 2 years with child mask Frog Item no. 035G2040	Above 4 years without mask Item no. 035G1000
		

Specialists in effective inhalation



How to use **PARI**chamber?

Felix the Frog shows and explains it to Mary the Ladybug.

1



First take off the cap of your spray, the metered dose inhaler (MDI).

Mary doesn't forget to shake the spray before using it.

2



Then you put the spray into the green connecting ring of PARIchamber.

If you are using a mask connect it to the mouthpiece.

3a



Put the mask to your face and make sure that the mask closes around your nose and mouth.

3b



When you get older, Mary, you won't need a mask any longer as you can use the mouthpiece. Then you have to make sure that you place the PARIchamber mouthpiece between your teeth and close your lips around the mouthpiece.

4



Now, Mary, you can press the spray once with your finger. Make sure to breathe in slowly and deeply.

5



If the doctor prescribed two or more puffs per application, you have to repeat step 3 and 4.

Don't forget!

If you are younger than 18 months, you should inhale 4 to 6 times. If you are older than 18 months, you should inhale 2 to 4 times.

Once you are finished, make sure to clean PARIchamber.

1

Remove the connecting ring and the mask if it is attached.



Cleaning:

2

Clean all parts gently but thoroughly in warm tap water with a little washing-up liquid. Rinse all parts thoroughly with fresh, clean water.



Disinfection:

3

PARIchamber including the mask can be disinfected in a steam disinfectant.



4

As an alternative, place all the parts in boiling water for at least 5 min. Make sure that PARIchamber is completely immersed.



Drying:

5

Allow PARIchamber to dry at room temperature or dry it with a clean tea towel.



6

Make sure to store PARIchamber safely in a dry, dust-free place.

